SilverCloud MB23

Electronic arm blood pressure monitor/Електронен апарат за измерване на кръвно налягане/Elektronisches Arm-Blutdruckmessgerät/Tensiometro electrónico de brazo/ Tensiomètre électronique au bras/ Elektronikus kar vérnyomásmérő/ Sfigmomanometro elettronico da braccio/ Elektronische armbloeddrummeter/ Elektroniczny ciśnieniomierz na ramię/ Tensiometru electronic de brat

EN	User manual	3
BG	Ръководство за употреба	9
DE	Benutzerhandbuch	16
ES	Manual de usuario	22
FR	Manuel utilisateur	28
HU	Használati utasítás	34
IT	Manuale utente	40
NL	Handleiding	46
PL	Instrukcja obsługi	52
RO	Manual de utilizare	58

Warnings

- Read this manual carefully to understand the instructions for use, performance and limitations of this device. Keep the manual for future reference.
- Do not use this device for self-diagnosis. Based on the results obtained with this device, do not decide by yourself to change the medication recommended by your doctor. If you have any doubts about the result, consult a doctor.
- This device has delicate, high-precision components. Therefore, avoid exposure to high temperatures or humidity, dust or smoke. The manufacturer guarantees the accuracy of this device only if it is stored and used properly.
- Do not attempt to calibrate or repair this unit by yourself. If you have problems with its operation, contact a specialized service.
- To clean this device, use a soft, dry cloth or lightly soaked in water or a mild detergent.
- If you have not used the appliance for more than 6 months, replace the batteries. It is recommended to use alkaline batteries.

Recommendation

The SilverCloud MB23 electronic arm blood pressure monitor is intended for use by people who are at least 18 years old and by medical staff to monitor blood pressure (systolic and diastolic) and pulse.

Before first use

Make sure that the appliance has 4 AA 1.5 V alkaline batteries correctly installed.

Before use, attach the cuff to the main unit.


If the icon indicating the discharged batteries is displayed on the screen, proceed as follows:

- Remove the cover from the battery compartment.
- Position the batteries respecting the indicated polarity.
- Replace the cover.

Notes:


- Using insufficiently charged batteries could affect the accuracy of the results.
- Do not use 1.2 V rechargeable batteries. Using them may cause inaccurate results.
- If the appliance has not been used for more than 6 months, replace the batteries. Even if the batteries are discharged or damaged or have been replaced, the device retains previous measurements.

Device description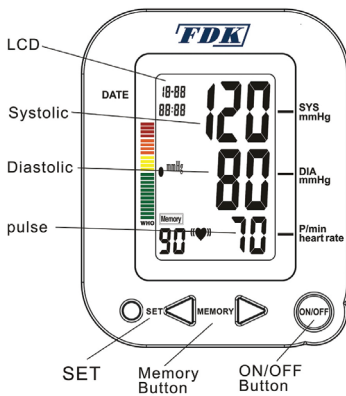

- SET button - Set time, day and month/activate or deactivate voice prompt
- MEMORY button - Display stored measurements/adjust time, day and month
- ON/OFF button - on/off the device
- LCD screen
- Systolic pressure indicator
- Diastolic pressure indicator
- Pulse indicator

Icons descriptions

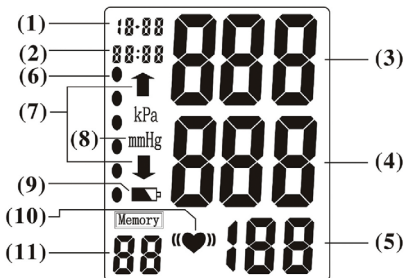1. Date: Month-Day
2. Hour: Hour-Minute
3. Systolic blood pressure (mmHg)
4. Diastolic blood pressure (mmHg)
5. Pulse (beats per minute)
6. WHO classification indicator
7. Cuff inflation/deflation indicator
8. Measurement unit of blood pressure
9. Low battery indicator
10. Irregular heartbeat indicator
11. Memory number

Correct blood pressure measurement

- Relax for 5 minutes before measuring your blood pressure.
- Do not smoke or drink coffee for at least 30 minutes before measuring your blood pressure.
- Do not move or speak during the measurement.
- Position the cuff directly on the bare arm.
- The cuff should be positioned on the left arm, above the elbow joint, at the heart level.
- The cuff should be neither too tight nor too light. There should be enough space between the cuff and the arm to fit two fingers.
- Lay your arm on the table with your palm facing up.
- After you are in a comfortable position, press the ON/OFF button on the device to turn it on.
- After 00 appears on the screen, the measurement starts.

Irregular heartbeat indicator

If the device detects irregular heartbeats, the icon (♥) will flash on the screen.

Setting the date, time and voice prompt activation/deactivation

- When the unit is turned off, press and hold the SET button. The year will flash on the screen. Use the MEMORY buttons (left arrow, right arrow) to reach the desired number.
- Press the SET button again. The month will flash on the screen. Use the MEMORY buttons (left arrow, right arrow) to reach the desired number.
- Press the SET button again. The day will flash on the screen. Use the MEMORY buttons (left arrow, right arrow) to reach the desired number.
- Continue the procedure for adjusting the hour and minutes. Note: The time is in 24h format.
- After setting the date and time, briefly press the SET button again to activate/deactivate the English voice prompt. SP ON or SP OFF will be displayed on the screen.
- To exit the date, time and language setting mode, press the SET button once. The screen will no longer display anything.

- If you replace the batteries, the date and time must be reconfigured.

Display of the last 3 measurements stored in the device

- When the unit is closed, press one of the MEMORY buttons (left arrow or right arrow).
- The AUg symbol will be displayed on the screen in the upper left corner.
- The voice prompt will read the values of the last measurement performed.
- Press one of the two MEMORY buttons again to move to the next saved measurement.
- The date and time of the measurement appear on the screen.

Notes:

- The measured values are automatically saved in the device.
- If no measurement is saved in the device, MEMORY NO will appear on the screen.
- To delete saved memories, press and hold one of the MEMORY buttons (left arrow or right arrow) until MEMORY NO appears on the screen.

Assessment of blood pressure for adults

The following standards for assessing blood pressure, regardless of age or gender, have been established taking into account WHO (World Health Organization) standards. Other risk factors such as diabetes, obesity, smoking, etc. must be considered. Always consult your doctor for a correct evaluation of the results.

Blood pressure classification	Tas (mmHg)	Tad (mmHg)	Chromatic indicator
Optimum	< 120	< 80	Green
Normal	120-129	80-84	
Normally high	130-139	85-89	
HTA stage 1	140-159	90-99	Yellow
HTA stage 2	160-179	100-109	Orange
HTA stage 3	≥ 180	≥ 110	Red

Technical specifications

- Measurement method: oscillometric
- Automatic inflation and deflation with air pump
- Measuring range: Pressure: 40 ~ 280mmHg/Pulse: 30 ~ 160 beats per minute
- Accuracy: Pressure \pm 5mmHg/Pulse \pm 5%
- Power supply: 6V DC (4 AA batteries)
- Autonomy: approx. 250 uses (180mmHg, once a day, 22 ° C)
- Prompt voice in English
- Memory: 90 measurements including date and time
- Cuff dimensions: 220mm ~ 360mm (+/- 5mm)
- Unit dimensions: 122 X 84 X 153 mm
- Weight: 284 gr (without batteries, without cuff)
- Working temperature: -26°C~ +60°C

Troubleshooting

Problem	Possible cause	Solution
The device does not perform measurements. Values are too high or too low.	The pneumatic system is blocked or the sleeve is too tight.	Make sure you fasten the cuff correctly and repeat the measurement.
	Your blood pressure is unstable.	Repeat the measurement. Stay calm. Do not move or speak during the measurement.
	The sleeve is not positioned correctly.	Position the cuff at heart level.
The irregular heartbeat symbol appears on the screen.	Irregular heart rhythm.	Wait 5 minutes and then repeat the measurement. If the symbol still appears on the screen, consult a doctor.

Poor sound of the voice processor.	The batteries may be discharged.	Replace the batteries.
The device does not work properly.	Interference during swelling of the sleeve or incorrect use of the appliance.	Reread the paragraph Making the measurement correctly
ER U appears on the screen	Insufficient swelling.	Wait for 5 minutes. If the appliance continues to operate abnormally, contact your dealer.
ER H appears on the screen	Swelling over 305 mmHg	
Er 1 appears on the screen	The device does not detect the pulse	
Er 2 appears on the screen	interference	Eliminate the cause of the interference
Er 3 appears on the screen	The measurement result is incorrect	Repeat the measurement.

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