# SILVERCLOUD RESPIRO 300 PRO

Compressor nebulizer / Компресорен пулверизатор / Kompressor-Zerstäuber /  
Nebulizador compresor / Nébuliseur à compresseur / Kompresszoros porlasztó /  
Nebulizzatore con compressore / Nebulizator sprężarki / Aparat de aerosoli

EN	User manual .....	2
BG	Ръководство за потребителя .....	5
DE	Benutzerhandbuch .....	8
ES	Manual de usuario .....	11
FR	Manuel utilisateur .....	14
HU	Használati utasítás .....	17
IT	Manuale utente .....	20
PL	Instrukcja obsługi .....	23
RO	Manual de utilizare .....	26

## INTRODUCTION

SilverCloud Respiro 300 Pro is a compact medical device designed to efficiently apply a physician prescribed medication to the bronchial lung passages. With proper care and use, it will provide you with many years of reliable treatment .

This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through the clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

This Compressor nebulizer should be used under the supervision of a licensed physician or a respiratory therapist. We encourage you to thoroughly read product's manual. Other than its intended use should always be avoided.

## WHAT'S IN THE BOX?

Nebulizer	Air tube	2 x mask	Nebulizer cup	Mouthpiece	5 x filter
					

## IMPORTANT SAFEGUARDS

Note: Read all instructions carefully before use.

Caution: Failure to read and observe all precautions could result in personal injury or equipment damage.

This unit is not suitable for use in anaesthetic breathing systems.

This unit is not suitable in suspension or high viscosity form. In such cases information should be sought from the drug supplier.

The following basic precautions are needed when using an electrical product:

- To avoid electrical shock: Keep the unit away from water. Do not immerse the power cord to the unit in liquid. Do not use while bathing.
- Never operate the unit if it has any damaged parts ( including power cord ), if it has been dropped or submersed in water. Promptly send it to a service center for examination and repair.
- The unit should not be used where flammable gas
- Disconnect the unit from the electrical outlet before cleaning.
- Do not use other attachments unless recommended by manufacturer
- Connect this product to an appropriate voltage outlet for you model
- Do not run this product unattended
- Always unplug the product immediately after use
- Never block the air openings of the main unit.

## OPERATION INSTRUCTIONS

*Note: Prior to initial operation, the nebulizer should be thoroughly cleaned.*

- Place your compressor Nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated.
- Open the clear cover and remove the inside accessories. Important: Prior to initial operation, the nebulizer should be thoroughly cleaned
- Gently twist the top part of the nebulizer counter-clockwise to disassemble the nebulizer.
- Fill the bottom section of the nebulizer with the medication prescribed by your physician. The medication cup can not be filled beyond the maximum level (6 c.c.)
- Gently twist the top part clockwise to reassemble the nebulizer. Be sure the two sections fit well.
- Attach one end of air tube to the base of the nebulizer
- Attach the other end of air tube to the air tube connector located on the front of the compressor.
- Attach mouth piece or mask per your choice to the top section of nebulizer.
- Plug the power cord into an appropriate electrical outlet. Push O/I button.

**Important:**

The compressor motor has a thermal protector which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:

- Switch off the unit. Use the button O/I
- Unplug the unit from the electrical outlet
- Wait about 30 minutes for the motor to cool down before another treatment. Make sure the air openings are not obstructed
- After 30minutes, restart your treatment.

*Note: When using this product near TV, microwave oven or other strong electric field, it could be disturbed. Avoid these situations.*

## CLEANING THE DEVICE

It is recommended that the nebulizer, mouthpiece and mask are thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions..  
Disconnect the air tube, nebulizer, mouthpiece and mask

Prepare one part white vinegar with three parts distilled water. Make sure mixed solution is enough to submerge the nebulizer, mouthpiece and mask.

Wash nebulizer, mouthpiece and mask in warm water and a mild detergent. Then wash them in hot tap water.

Submerge these parts in the vinegar and water solution for thirty minutes.

Complete rinsing

### Filter change

Do not use cotton or any other materials

Do not wash or clean the filter.

Only use filters supplied by your distributor.

Do not operate without a filter.

Change the filter every 30 days or when the filter turn gray.

### Filter changing procedure:

Remove the filter cover

Replace the used filter with a new one

Replace the filter cover.

## TECHNICAL SPECIFICATIONS

Power voltage	230 V / 50Hz
Current	0.7 A
Medication capacity	6 ml
MMAD (Mass Median Aerodynamic Diameter)	3 µm
Sound level	58 dBA
Average nebulization rate	> 0.33 ml/minute
Compressor pressure range	30 - 36 Psi (210 - 250 Kpa / 2.1 - 2.5 bar)
Operating pressure range	8 - 16 Psi (50 - 100 Kpa / 0.5 - 1.0 bar)
Appropriate medication quantities	min. 2 ml, max. 6 ml
Nebulization rate	0.33 ml/min
Aerosol output	0.33 ml (2ml, 1%NaF)
Aerosol output rate	0.06ml/min (2ml, 1%NaF)
Working temperature	-20 ~ +55°C
Dimensions	270 x 200 x 110 mm
Weight	2 kg

## TROUBLESHOOTING

Nothing happens when the power switch is pressed

- Check that the plug is inserted in an electrical outlet

No nebulisation or low nebulisation, when the power is on

- Add the correct amount of medication to the medication tank (2 ml - 6 ml)
- Install all the accessories correctly
- Make sure that the nozzle is free of blockages
- Make sure that the nebuliser kit is not tilted at an angle of more than 45 degrees
- Replace the filter

The device is abnormally loud

- Attach the air filter cover correctly

The device is very hot.

- Do not cover the compressor with any type of cover during use; - Limit use to 20 minutes at a time, and allow a 30 minutes interval before using the device again.

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